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en – Surgical guide manual

STORAGE CONDITIONS

The surgical guide is composed of a material that it sensitive to humidity and light (UV radiations), as such we recommend you to:

HYGITECH

smart dental surgery solutions

- Keep the surgical guide in the protective package in which it was delivered and must be maintained within as long as possible until the surgery.
- Store the surgical guide in dry place and protected from light (especially direct solar light)
- Never expose the guide to liquid (water or other) during more than 30 minutes because there is an important risk of deformation.

OPERATING PROTOCOL STEP BY STEP

1 - Disinfection of the surgical guide

The surgical guide does not support autoclave sterilization, only disinfection is possible. Dip the guide into chlorhexidine for 20 minutes, dry it quickly without using a heat source but with a sterile, lint-free cloth.

2 - Disinfection and sterilization of the pilot drill

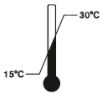
at 134°C for 18 minutes in an autoclave.

3 - The whole drilling process is completed under local anesthesia

The injections of the anesthetic products must be made at a distance from the guides support area. In fact, the anesthesia stage is a form of misplacement compared to the initial project, the local injection of the anesthetic product creates mucosal swellings sometimes unnoticeable but which can hinder the insertion of the tooth or mucous supported guide, or cause its placement to be misaligned, which can induce great instability of the guide.

4 - Implementation of the guide

The guide must be correctly implemented, it is a key step for the successful completion of the operation.

Conserver entre 15°C et 30°C
Keep between 15°C and 30°C

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